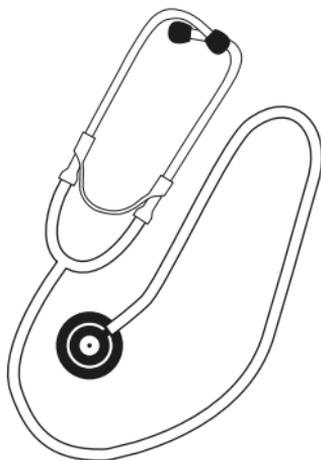
WESTMED

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Please read these instructions for use carefully and completely before use, and observe the care instructions.

The stethoscope is a diagnostic instrument and consists of an earpiece, a tube system, and the chestpiece. It is used to auscultate (listen to) heart sounds, lung sounds, and other audible vital parameters. Depending on the examination, the stethoscope is to be placed on the desired body region and used under slight pressure. In the case of double-sided stethoscopes, the input channel (funnel or diaphragm) can be selected by switching over. Observance of all the information in these operating instructions and the relevant regulations and guidelines is part of the intended use. Any other use is not as intended, and the manufacturer is not liable for any resulting damage!

This product may only be used by authorized specialist personnel!

Intended patient group

Adults

Delivery contents

1. Sound funnel with large and small diaphragm
2. Chromed ear hooks with ear olives
3. Sturdy tubing

Safety instructions

- Before each use, check the stethoscope for proper function and cleanliness. Do not use the product if you notice any damage or soiling.
- Never fit the stethoscope without ear olives to avoid injury to the ear canal! Damaged ear olives must be replaced before use.
- Always ensure that the new parts are firmly seated after replacement. Only use the original spare parts and accessories supplied with the stethoscope.
- Never place the stethoscope in liquids.
- The stethoscope is not suitable for sterilization processes!
- Ambient noise can interfere with use.
- Keep/protect the stethoscope away from high temperatures, strong solvents, and oils.

Operation

- First, the binaural earpieces are placed in the ears.
- To check that they are working properly, gently tap the diaphragm with your finger.
- If you do not hear a tapping sound, there is a defect.
- In case of audible knocking, you can start investigation.

Cleaning and disinfection

After each patient, the stethoscope should be cleaned and disinfected with a 70% alcohol solution (e.g., with soaked wipes). Observe the exposure times of the product used. Furthermore, the individual components of the stethoscope should be cleaned thoroughly on a regular basis. Use a 70% alcohol solution or a mild soap solution for this purpose. The ear olives and the membrane can be removed for thorough cleaning. For the membrane, pull off the retaining ring. Handle the diaphragm carefully to avoid damage! Do not reassemble the stethoscope until all parts are dry.

Disposal

The product must be disposed of in accordance with local regulations.

Storage and transport

Store and transport the stethoscopes protected from dust, moisture, and contamination.

Maintenance and service

The end of product life is normally determined by wear and damage from use. Damaged parts must be replaced! For possible repairs, please contact your specialist dealer.

Warranty

If the product is handled properly and our instructions for use are observed and original parts are used, the statutory warranty of two years commencing on the date of sale applies. Sounds and heart murmurs of infants can best be perceived with this bell.

No liability is accepted for translation errors.

All information without guarantee – subject to change.

Art. No.: 5810011
Manufacturer No.: HS-30B

MDR (EU) 2017/745

EN ISO 14971:2012; ISO 10993-5:2009; ISO 10993-10:2010
EN ISO 13485:2016; EN ISO 15223-1:2016

WESTMED distributed by
Fleischhacker GmbH & Co. KG
An der Silberkuhle 18, 58239 Schwerte, Germany
Tel.: +49 2304 9310, Fax: +49 2304 931199
info@fleischhacker.biz

HONSUN (NANTONG) Co., Ltd
No. 8 Tongxing Road Economic & Technological
Development Area, 226009 Nantong City,
Jiangsu Province, P. R. China

Shanghai International Holding Corp. GmbH (Europe)
Eiffestraße 80, 20537 Hamburg, Germany
SRN: DE-AR-000000001