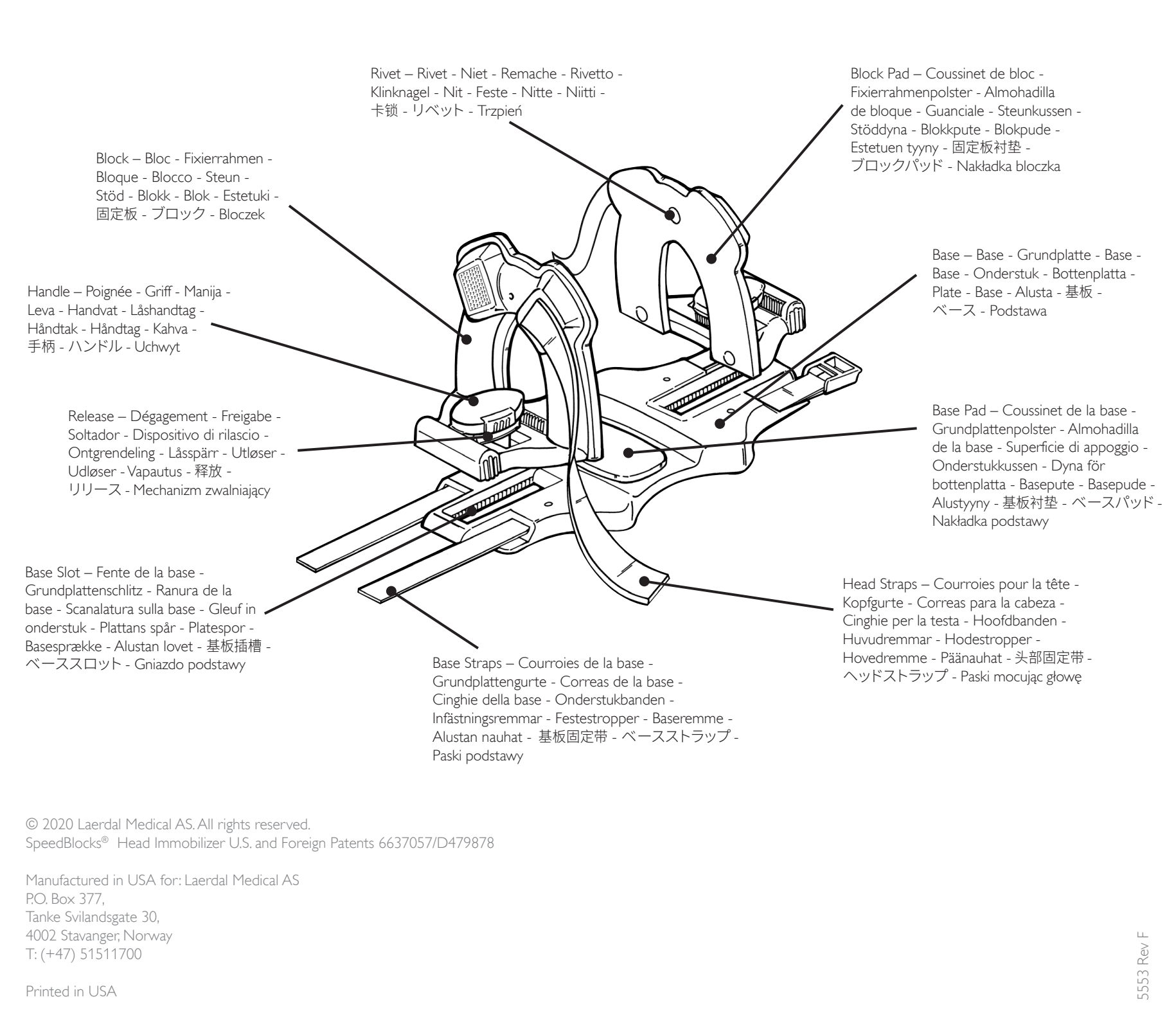
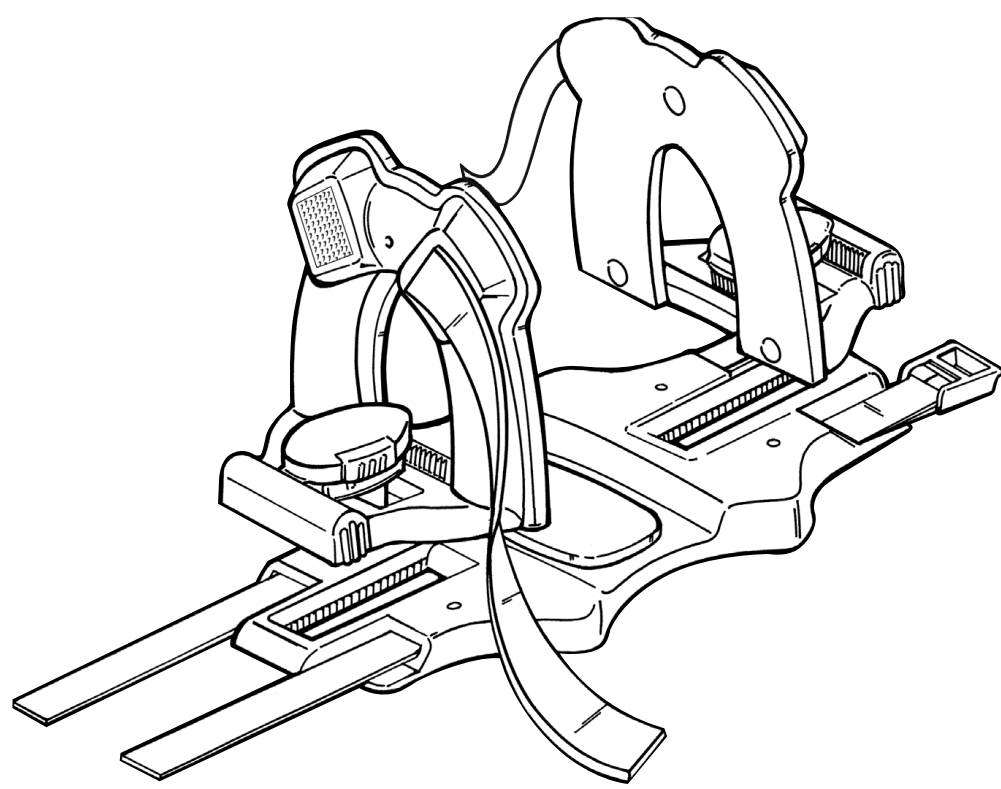
SpeedBlocks Head Immobilizer

User Guide

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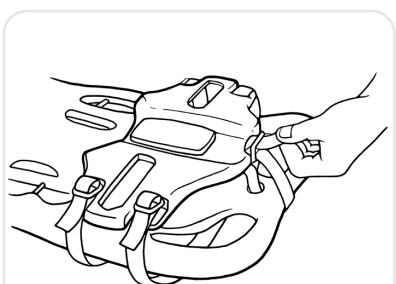
- Cautions:**
- The SpeedBlocks Head Immobilizer is intended for patients two years of age or older and is not compatible with vacuum mattresses.
 - SpeedBlocks should only be used by persons who have received adequate training.

Connect base to spineboard using the two long straps and one short strap.

Preparation

- If your board has no center hole, use the 2 long straps. Check all straps for tightness before each use.
- Place patient on spineboard with the patient's head centered on the base. Additional padding (PadPack™ Alignment Pads) under the head of an adult or torso of a child may be necessary to maintain neutral alignment.
- Remove tape liner on new base pad before applying. Remove soiled block pads and attach new ones by stretching foam over rivet. Attach new head straps if necessary.
- Keep a spare set of pads and head straps with your spinal immobilization equipment.

Application

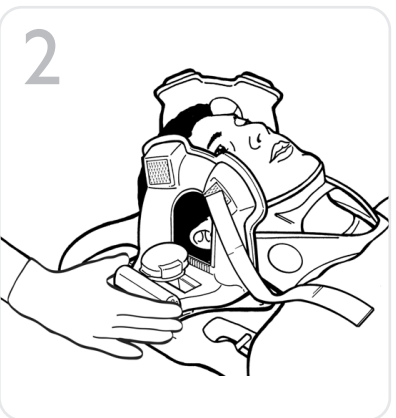
Maintain manual immobilization while applying head immobilizer to patient. (Not shown in images 1-4 for simplification)

Connect Blocks To Base


Squeeze the release and lift each handle. Insert the blocks into the base slots.

Adjust Block Position


Move blocks against the head and align the block openings to the patient's ears.

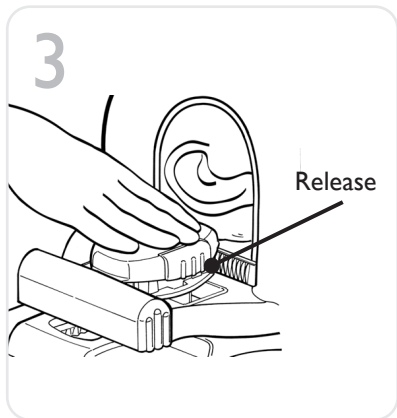Lock Blocks In Place

Push handle down to lock. To re-adjust block position, squeeze the release and lift the handle.

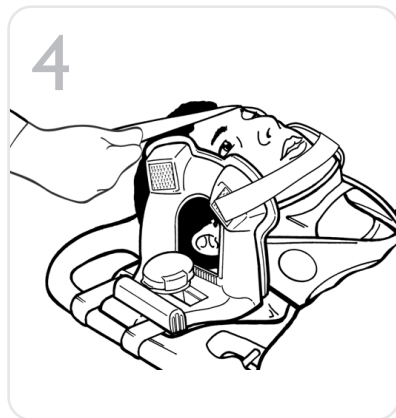Fasten Head Straps

Fasten each strap to the opposing block. Center the straps over the chin and forehead.

Cleaning and Preparation for Re-Use

- Separate Parts:** Disconnect blocks from base and remove base from spineboard. Discard any block pad, head strap, or base pad that has been contaminated with body fluids. With proper cleaning, the base and blocks can be reused.
- Scrub & Soak:** Scrub parts in 1:61 100 °F (38 °C) water and dishwashing detergent solution for 4 minutes. Rinse parts in clean 100 °F (38 °C) water. Soak parts in 100:1 (0.5% concentration) 100 °F (38 °C) water and household bleach solution for 10 minutes.

Warranty

Refer to the Laerdal Global Warranty for terms and conditions. For more information visit www.laerdal.com.

CE	This medical device complies with the general safety and performance requirements of Regulation (EU) 2017/745 for medical devices.	MD	Medical Device
-----------	--	-----------	----------------

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