# Compat® Gravity


## COMPAT® GRAVITY SETS

إرشادات الاستخدام / Návod k použití / Brugsanvisning / Gebrauchsanweisung / Instructions for use /  
Instrucciones de uso / Käyttöohje / Mode d'emploi / Istruzioni per l'uso / Gebruiksaanwijzing / Bruksanvisning /  
Instrukcja użycia / Instruções de uso / Návod na použitie / Bruksanvisning

www.compat.com


### COMPAT® GRAVITY UNIVERSAL SET

REF 12298230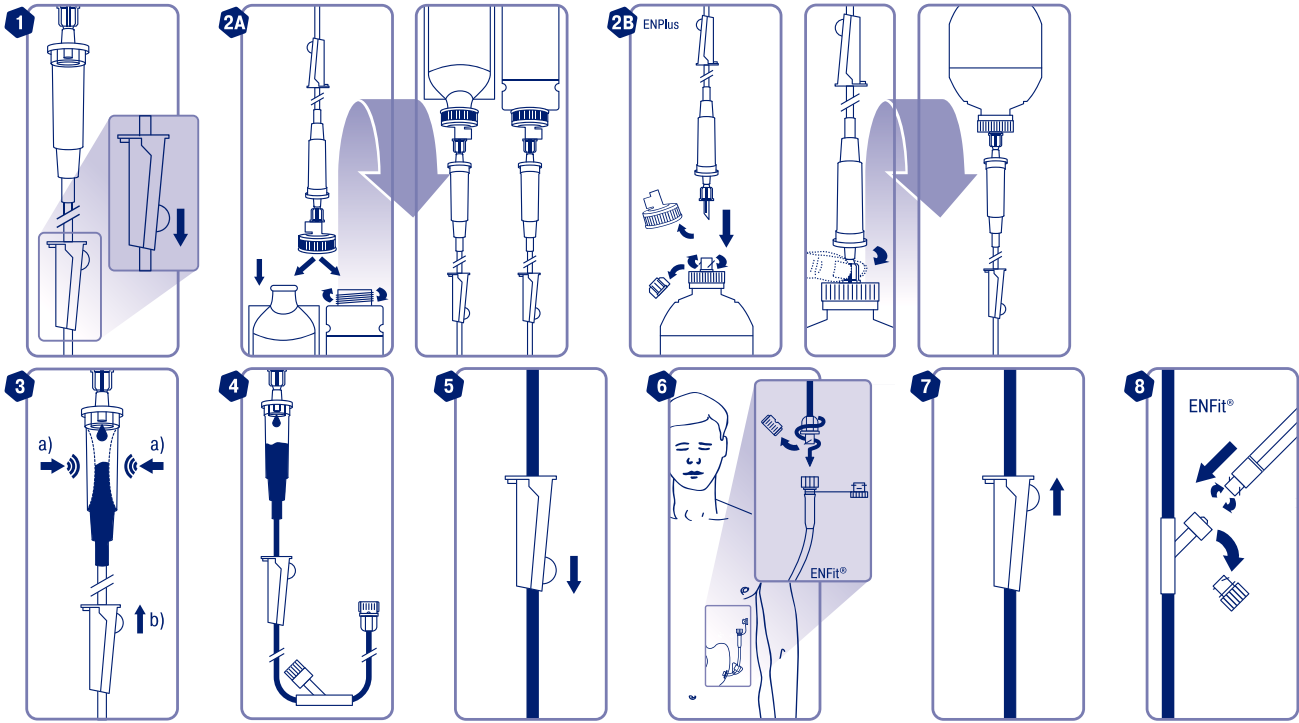


### COMPAT® GRAVITY COMBISET 1.5 L

REF 12298204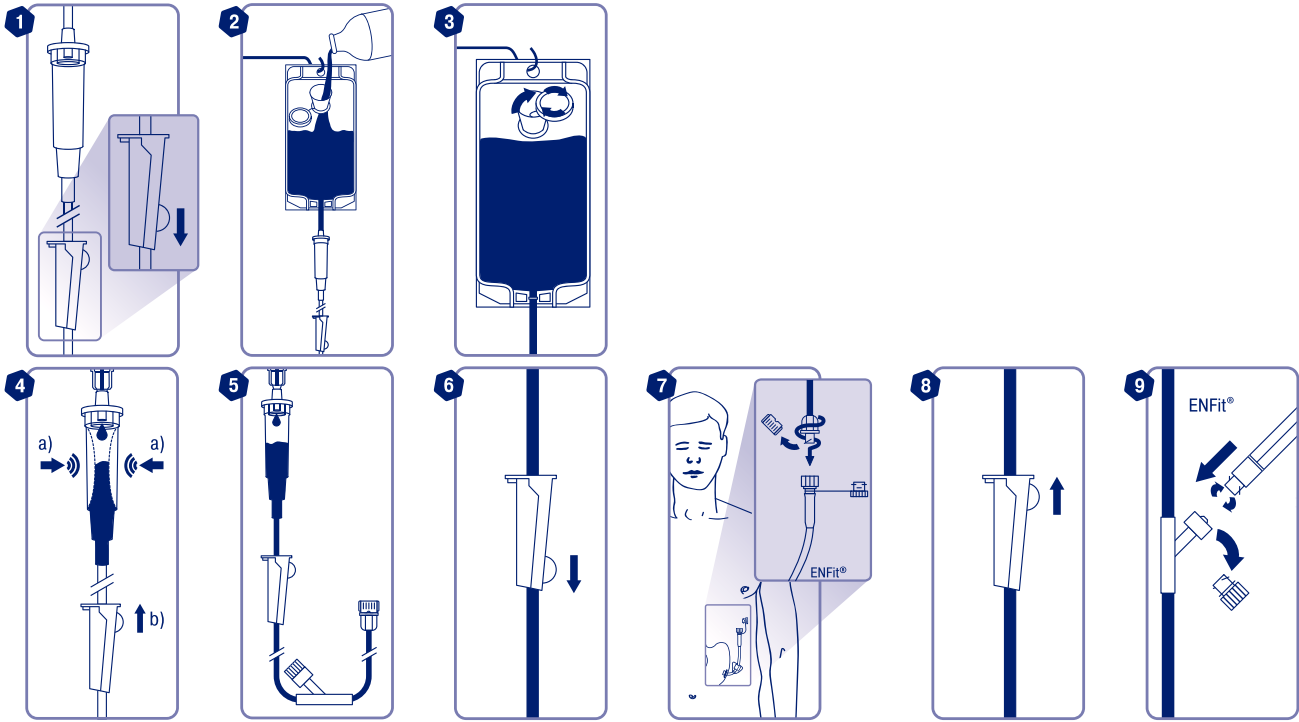

### COMPAT® GRAVITY SPIKE SET

REF 12298205

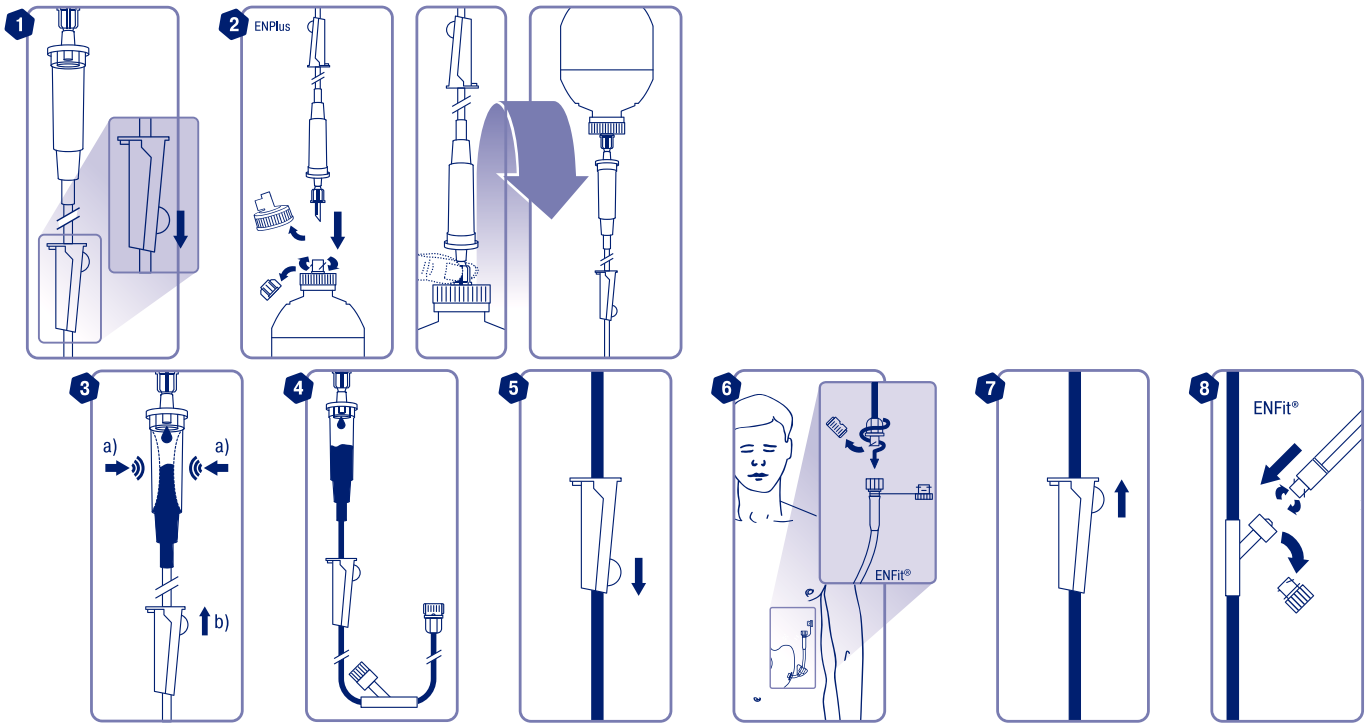© Unless stated otherwise, all trademarks are used with permission.

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**🇬🇧 Enteral giving sets for tube-feeding. For gravity use. 🔗 DEHP-free. Important Notes:** Do not use for more than 24 hours. For enteral use only.
🔗
Not for intravenous use. Do not use with an enteral feeding pump.
🔗
Do not use if package is damaged. Any reprocessing (especially sterilization) will compromise the structural integrity of the device.
**Please Note:** The giving set may become a source of infection and have altered performance after use and in case of reuse. Follow good clinical practice and all local and national regulations for transportation and disposal. Small parts are a potential choking hazard if this device is not used as intended. Use under adult supervision.
**Additional Information:** Connect the enteral giving set to the enteral feeding tube and ensure it is properly connected before feeding. Always monitor patient during feeding. Make sure the clamp is closed and the enteral giving set is disconnected from the feeding tube when not feeding. To obtain further information please contact your sales representative or go to www.compat.com.

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