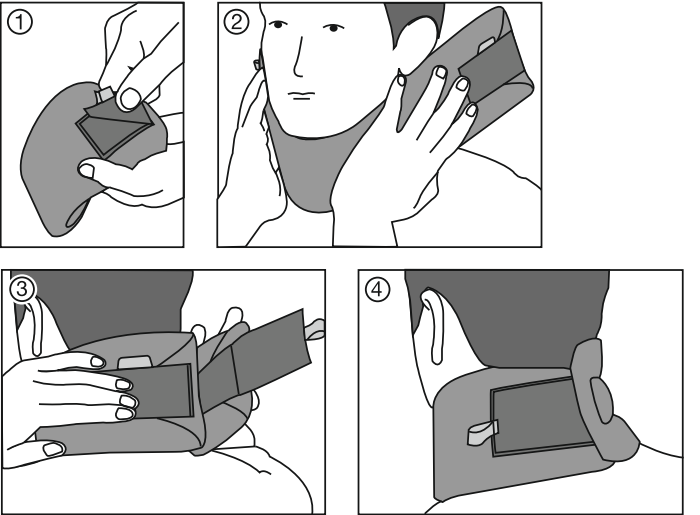
de

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Instructions for use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.

Product description and performance characteristics:
Cellacare® Cervical Classic is an anatomically shaped orthosis for stabilisation of the cervical spine. Neck muscle tension is decreased by keeping the musculature warm.

Product composition:
Polyurethane, cotton, elastane, polyamide, polyester

Intended purpose:
Orthosis for support and stabilisation of the cervical spine.

Indications:

- Cervical spine pain syndrome (cervical syndrome, cervical brachial syndrome)
- Muscular torticollis
- Osteoarthritis of the cervical spine without neurological malfunction
- Dysfunction of the vertebral joints (hypomobile dysfunction)
- Spondylitis ankylosans (M. Bechterev)
- Discectomy (temporarily after surgery)
- Distortion of the cervical spine (temporarily after trauma)

Contra-indications:

- Known allergy and / or hypersensitivity to any of the product components
- Severe neck injuries / disorders such as:
 - Instability/fragility of the cervical spine with vertebral fractures, in case of bone tumours or metastases or advanced osteoporosis
 - Cervical spine disorders with neurological deficits (e.g. disc prolapse, protrusion, facet syndrome)

Side effects:
In very rare cases, skin irritations and/or allergies may occur.

Warnings and precautions:

- Do not use Cellacare® Cervical Classic unless directed by a health care professional. The first time patients use the product they need to have a health care professional show them how to put it on correctly.
- The health care professional will determine how long the product is to be worn per day, based on the indications and healing progress.
- Cellacare® Cervical Classic shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before application of the product.
- In case of skin diseases and / or injuries in the application area (open, non-intact or injured skin) please contact your treating physician / seek medical advice before application of the product.
- Apply the product in accordance with the instructions for use and illustrations provided. Please check the positioning of the product during use and correct as necessary.
- In case of discomfort or problems, consult a health care professional.
- If local swelling or numbness occurs in the part of the body being treated, the product must be removed and a medically qualified health care professional consulted.
- Local swelling or paraesthesia / sensation of numbness may occur in the arms in the area supplied by the cervical spinal nerves when Cellacare® Cervical Classic is applied too tightly (the hook-and-loop fastener makes it possible to adjust the tightness).

- Cellacare® Cervical Classic is intended for use on a single patient only.
- The product’s durability may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook-and-loop fastener. To prevent this, the hook-and-loop fastener should always be fastened, for example when the product is not being worn or when being washed.
- Cellacare® Cervical Classic must not be used beyond the last day of the month indicated on the packaging (expiry date).
- If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

Usage information:

Preparation:
Open the hook-and-loop fastener of Cellacare® Cervical Classic (Fig. 1).

Usage:

1. Put Cellacare® Cervical Classic around the neck with the chin cutaway under the neck (Fig. 2).
2. Close the hook-and-loop fastener at the back of the neck (Fig. 3).
3. Cellacare® Cervical Classic should support, and not constrict, the neck (Fig. 4).

Care recommendations:
Wash the orthosis at 30 °C (delicate cycle) with like colours. Do not bleach. Allow Cellacare® Cervical Classic to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean. Fasten the hook-and-loop closure and if possible use a mesh bag. Clean the hook-and-loop closure regularly so it retains its ability to fasten tightly, and to preserve the performance of the product. The additional cover can be washed separately.

Storage and transport conditions:
Cellacare® Cervical Classic must be kept away from light and moisture (preferably in the original packaging).

Disposal:
In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

General information:
In case of a serious incident, contact the manufacturer and the responsible health authorities.

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