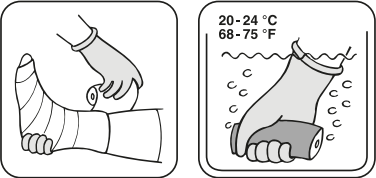
# Cellacast® Active

Polyester Stützverband  
Bandage de soutien en polyester  
Polyester casting tape  
Venda estabilizadora de poliéster  
Ligadura de descanso de poliéster  
Bandaggio di sostegno in poliester  
Polyester steunverband  
Polyester støtteforbinding  
Polyesterstöd förband  
Avstøpningstape av polyester  
Polyesteritukiside  
Polyesterový podpůrný obvaz  
Polyesterová stabilizačná bandáž  
Poliestrowy opatrunkowy uniuruchamiający  
Poliesterski podpirajući povoj  
Πολυεστερική ταινία εκμαγιών  
Polyester destek bandajı

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## en Instructions for Use

**Product description and performance features**  
Polyester casting tape

**Product composition**  
Knitted polyester fabric impregnated with polyurethane resin.

**Intended use**  
For the preparation of casts for mobilisation and targeted immobilisation.

**Indications**  
For treatment after fractures, surgery and tendon injuries.

**Contraindications**  
Known allergy and/or hypersensitivity to any of the product components.

**Side effects**  
In very rare cases skin irritations and/or allergies may occur.

**Warnings and precautions**

- Wear gloves during application.
- Only use the product in well-ventilated areas.
- The uncured cast can irritate skin and eyes and sensitisation by inhalation is possible.
- Avoid direct contact with the uncured cast. Any points of contact can be cleaned immediately with alcohol or acetone.
- Advise the patient that bathing with the cast is not recommended because wet padding material under casts can lead to skin maceration. Wet casts can be dried with a towel and a hair dryer.
- Advise the patient that he/she must report any feelings of numbness or pressure immediately and must ensure that no foreign bodies (e.g. sand) get under the cast because they can lead to skin irritation.
- Do not use if the packaging is damaged or inadvertently opened.

**Application instructions**  
**1. Preparation**  
Please note that direct contact with the uncured casting tape is to be avoided. Wear gloves during application. The patient's skin must first be covered with skin protection (e.g. tg® Tubular Bandage, tg® Soft) and padding (e.g. Cellona® Synthetic Undercast Padding, Cellona® Padding, Cellona® Cast Edge Padding). Only open and process one Cellacast® Active casting tape at a time because moisture in the air can lead to premature curing.  
**2. Application**  
• Immerse the casting tape at an angle in water with a temperature of 20°C to 24°C and lightly squeeze it 1 to 2 times to ensure that it is fully moistened. Remove the casting tape and shake off any excess water. Always use fresh water.

**Caution:** The temperature of the water in which the casting tape is immersed must not exceed 24°C because this can lead to an increase in the setting temperature which may burn the patient.

- Apply the Cellacast® Active casting tapes using circular turns with no tension and with each layer overlapping half the width of the underlying layer. Then mould the cast thoroughly. Ensure that no wrinkles develop.
- The setting time is 2.5–5 minutes. Cellacast® Active casts can bear weight after 30 minutes. The cured cast can be removed using conventional tools.

**3. Storage and transport conditions**  
Store Cellacast® Active casting tapes at a consistent temperature of no more than 30°C in a dry location.

**4. Disposal**  
In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (German Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems. When disposing of casting tape that has not been used, open the packaging so that the product can cure. After at least 5 minutes, the cured product can be disposed of in household waste.

**General information**  
For single use.  
In case of serious incident, contact the manufacturer and the responsible health authorities.

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