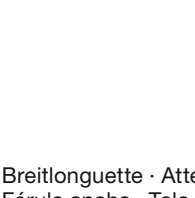
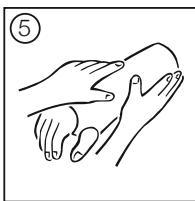
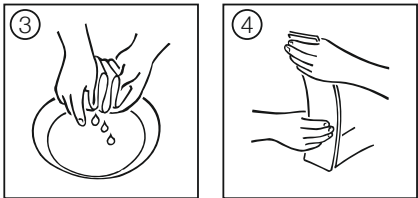
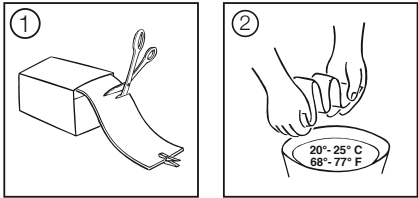
Cellona®

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MD Dispositif médical

en Instructions for use

Product description and features:
4-ly ready-to-dip Plaster of Paris splint
Ready-to-dip Plaster of Paris bandage to mould wide material

Product composition:
Cotton gauze coated with a mixture of α- and β-calcium sulphate hemihydrate

Intended purpose:
Cellona® P.o.P. Splints and Cellona® P.o.P. Wide Material are intended to be used for moulding of immobilising casts on injured or fractured limbs and for moulding of splints or reinforcement of circular casts.

Contraindications:
Known allergy and / or hypersensitivity to any of the product components.

Side effects:
In very rare cases skin irritations and / or allergies may occur.

- Warnings and precautions:**
- Advise the patient against taking a bath with the cast, as Plaster of Paris materials are not resistant to moisture.
 - Advise the patient that he/she should immediately disclose any sensation of numbness or pressure and must take care that no foreign bodies (e.g. sand) get under the cast, as this may lead to skin irritations.
 - Cellona® P.o.P. products shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before application of the product.

Instructions for use:

Cellona® Plaster of Paris Splint:

1. Pull the desired length from the dispenser pack and cut.
2. Push the cut length together like an accordion and briefly dip in water warmed to 20-25°C.
Warning: The temperature of the water must not exceed 25°C, as this may lead to an increase in the setting temperature, which may cause burns in the patient.
3. Remove the splint from the water and allow water to drip off. Then squeeze the splint lightly to remove excess water.
4. Smooth out the splint prior to use.

5. Apply the splint and then mould thoroughly to ensure a good bonding of layers.

Cellona® Wide Material:

1. Lay the required number of layers one on top of the other and cut the Wide Material to the desired length and shape.
2. Push the Wide Material together like an accordion and briefly dip in water warmed to 20-25°C.
Warning: The temperature of the water must not exceed 25°C, as this may lead to an increase in the setting temperature, which may cause burns in the patient.
3. Remove the Wide Material from the water and allow excess water to drip off. Then smooth out the Wide Material.
4. Apply the Wide Material and then mould thoroughly to ensure a good bonding of layers.

The setting time is 3-4 mins. Cellona® Plaster of Paris splints are partially load-bearing after 30 minutes and fully load-bearing after 36-48 hrs. The cured Plaster of Paris can be removed with conventional instruments.

Storage and transport conditions:
Store in dry place under room conditions.

Disposal:
In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance - AVV). Packaging labelled as recyclable should be taken into the respective national recycling systems.

General instructions:

- For single use.
- In case of a serious incident, contact the manufacturer and the responsible health authorities.

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