# Vliwasorb® adhesive

Superabsorbierender Wundverband, selbstklebend
Pansement super-absorbant, auto-adhésif

Superabsorbent Wound Dressing, self-adhesive
Apósito superabsorbente, autoadhesivo

Compressa superassorbente para feridas, autoadesiva

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**Product description**
Vliwasorb® adhesive is a flexible and soft, superabsorbent, self-adhesive wound dressing used for wounds with moderate to high amounts of exudate. Vliwasorb® adhesive consists of a superabsorbent polymer (sodium polyacrylate) which can bind high quantities of wound exudate, imbedded in cellulose, and a polyurethane support layer with an acrylate adhesive.

The wound dressing binds excess exudate and thereby protects the wound and wound environment against maceration.

The smooth, elastic polyethylene wound contact layer reduces the risk of adhesion to the wound.\*

**Product composition**
Polyethylene, polypropylene, cellulose, polyethylenterephthalate, ethylene vinyl acetate copolymer, green tea extract, polyacrylate, polyurethane

**Intended use**
Absorbent multilayer wound dressing indicated for exudate management of acute and chronic wounds.

**Indications**
Vliwasorb® adhesive is indicated for the treatment of wounds with moderate to high amounts of exudate:

- pressure ulcers, arterial ulcers, venous leg ulcers, diabetic ulcers
- postoperative wounds healing by secondary intention
- closed laparotomy wounds
- superficial 2nd degree burns

It can also be used as a secondary dressing with an appropriate wound filler (recommended are Suprasorb® Liqueal Hydroactive Filler Dressing, Suprasorb® A Calcium Alginate Dressing, Suprasorb® A + Ag Antimicrobial Calcium Alginate Wound Dressing or Opraclean® gauze Dressing with iodoform) for treating deep wounds.

**Contraindications**
Known allergy and / or hypersensitivity to any of the product components.

Do not use in tunnelling wound pockets as the product can expand considerably when wound exudate is absorbed. Do not use on wounds with light exudation, as the wound may dry out and suction pain may occur.

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